BAL-Series

BAL1-B2
BAL1-C2
BAL1-C3

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
Intended use

The BAL series instruments are used for endoscopic extraction of stones and sludge from the bile duct and pancreatic duct and to clear an occlusion of pancreatic duct after administration of contrast agent.

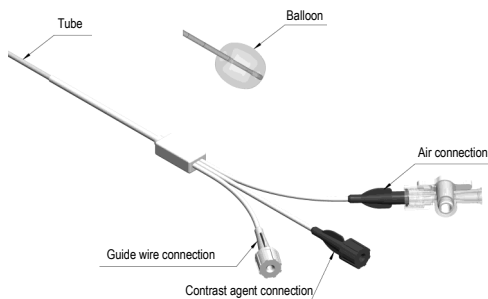Product features

Qualification of user

Use of these instruments requires comprehensive knowledge of the technical principles, clinical applications and risks of gastrointestinal endoscopy. The instruments should only be used by or under the supervision of physicians who are adequately trained and experienced in endoscopic techniques.

General instructions

Use this instrument only and exclusively for the purposes described in these instructions.

ⓘ ATTENTION! Instruments marked with this symbol are intended for single use only and are sterilised with ethylene oxide.

Sterile instruments can be used immediately. Before use, check the "Use by" date on the packaging, as sterile instruments are only allowed to be used up to this date.

Do NOT use the instrument if the sterile packaging has tears or perforations, sealing is not ensured or moisture has penetrated. All medwork instruments should be stored in a dry place that is protected from light. Keep all instructions for use in a safe and easily accessible place.

medwork instruments that are labelled as intended for single use only must not be reprocessed, resterilised or reused. Reuse, reprocessing or resterilisation can alter product properties, resulting in functional failure that can endanger the patient's health or lead to disease, injury or death. Reuse, reprocessing or resterilisation furthermore entails the risk of contamination of the patient or the instrument, as well as the risk of cross contamination, including the transmission of infectious diseases. Contamination of the instrument can result in illness, injury or death of the patient.

Indications

Cholelithiasis.

Contraindications

The contraindications for the BAL series are the same as the specific contraindications for endoscopic retrograde cholangiopancreatography.

Possible complications

Pancreatitis, cholangitis, perforation, haemorrhage, aspiration, infection, sepsis, allergic reaction in particular to latex, hypertension, hypotension, respiratory depression or arrest, cardiac dysrhythmia or arrest. Additional possible complications in conjunction with balloon stone extraction include impaction of the stone to be extracted, local inflammation or pressure necroses.

Precautions

In order to ensure that the examination proceeds smoothly, the diameter of the endoscope working channel and the diameter of the instruments must be matched to one another.

After removal from the packaging, check the instruments for correct function, kinks, fractures, rough surfaces, sharp edges and protrusions. If you discover any damage or malfunction on the instruments, do NOT use them and inform your responsible field service contact or our branch office.

Any person who markets or uses medical devices is required to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Attention! When using stone extraction balloons that are compatible with guide wires, observe the guide wire diameter recommended on the product label.

Instructions for use

Insert the balloon catheter into the working channel of the endoscope and gradually slide the catheter forward until the tip is visible in the endoscopic image. Before using a guide wire, we recommend you rinse the guide wire lumen with sterile saline solution to make sliding easier. For double lumen balloon catheters, the mandrin must be removed before placement. Probe the bile system until behind the stone. Fit the included disposable syringe to the marked connector and under radiological guidance fill the balloon with air, sterile saline solution or Aqua Dest. Please observe the maximum defined filling capacity (ml) on the marked connector for the product. Carefully pull the disposable stone extraction balloon toward the papilla. Strong resistance can point to a stone that is too large or impacted. There is a risk of damaging the balloon. Deflate the balloon before passing the papilla. Pull the instrument out of the endoscope slowly to avoid contaminating the patient and handler with blood and other bodily fluids and subsequent risk of infection.

After completion of the surgical procedure

Instruments intended for single use, including packaging, must be disposed of in accordance with the relevant applicable hospital and administrative guidelines and in accordance with the relevant applicable legal regulations.

List of all icons used on medwork products

	Date of manufacture		Use by
	Follow the instructions for use		Caution near pacemakers
	Applied part Type BF		Do not use if packaging is damaged
LOT	Batch code		Not suitable for lithotripsy
REF	Item number		Contains natural rubber latex
	Packaging unit		Gastroscopy
	Do not reuse		Colonoscopy
STERILE EO	Sterilised using ethylene oxide		Enteroscopy
	Non-sterile		ERCPC
	Manufacturer	MD	Medical device
	Sterile barrier system		

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