**FUJIFILM**

**medwork**

## BAS-Series

**BAS2-A2**

**BAS2-C1**

**BAS2-K2**

**CE**

404450305701

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## Instructions for use

### Intended use

The BAS series instruments are used for endoscopic removal of stones from the bile duct and pancreatic duct.

### Product features

Figure 1: E.g.: BAS2-A2-30-23-200


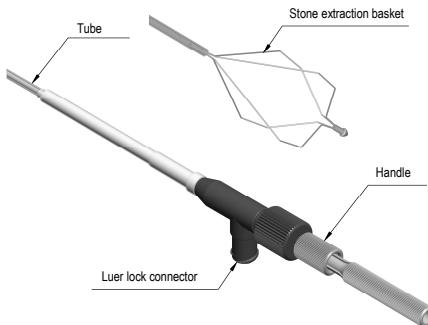
Figure 2: Disassembly and assembly
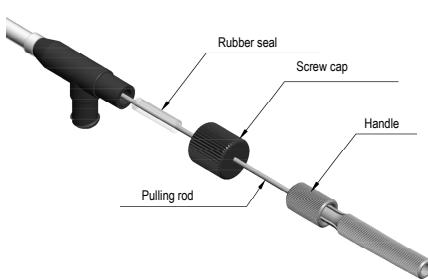
### Qualification of user

Use of these instruments requires comprehensive knowledge of the technical principles, clinical applications and risks of gastrointestinal endoscopy. The instruments should only be used by or under the supervision of physicians who are adequately trained and experienced in endoscopic techniques.

### General instructions

Use this instrument only and exclusively for the purposes described in these instructions.

All instruments labelled as unsterile must be prepared in accordance with the applicable medwork preparation instructions before use for the first time.

All medwork instruments should be stored in a dry place that is protected from light. Keep all instructions for use in a safe and easily accessible place.

### Indications

Choledocholithiasis.

### Contraindications

The contraindications for the BAS series correspond to the specific contraindications for oesophageal gastroduodenoscopy, colonoscopy, sigmoidoscopy, rectoscopy and endoscopic retrograde cholangiopancreatography.

### Possible complications

Perforation, cholangitis, haemorrhage, aspiration, infection, sepsis, allergic reaction to contrast medium or medications, hypertension, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest. Additional possible complications when extracting foreign bodies are the impaction of the foreign body to be extracted, aspiration of the foreign body, local inflammation or pressure necroses.

### Precautions

In order to ensure that the examination proceeds smoothly, the diameter of the endoscope working channel and the diameter of the instruments must be matched to one another.

After removal from the packaging, check the instruments for correct function, kinks, fractures, rough surfaces, sharp edges and protrusions. If you discover any damage or malfunction on the instruments, do NOT use them and inform your responsible field service contact or our branch office.

Any person who markets or uses medical devices is required to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**ATTENTION!** For stone extraction baskets of the BAS series, only system components of the LIT series may be used, medwork assumes no product liability if the product is used with other lithotripsy system components.

All BAS series stone extractors are suitable for mechanical lithotripsy, with the exception of the stone extractors marked with this icon.

### Instructions for use

Insert the retracted basket into the working channel of the endoscope and advance the stone extraction basket little by little until the catheter tip is visible in the endoscopic image. Probe the desired duct system until the basket has passed the stone and carefully capture the stone under radiological guidance. The stone extractor is retracted and extended by operation of the clamping grip. Small stones can be extracted by pulling back the catheter via the papilla. Slowly pull the instrument out of the endoscope to prevent contamination of the patient and user with blood and other body fluids, and an associated risk of infection.

**Attention!** If the stone cannot pass through the papilla, the basket must be prepared for mechanical lithotripsy.

### Non-endoscopic mechanical lithotripsy under radiological control:

Screw on the clamping grip and pull it from the pulling rod. Unscrew the screw cap and remove the rubber seal. Carefully pull the tube out of the endoscope over the traction cable. Then remove the endoscope and the tube.

Use the lead line of the stone extraction basket to slide the flexible emergency spiral (LIT1-N4-38-090) forward to the stone. Make sure that the basket is continuously under tension if possible.

Insert the traction cable into the lithotriptor handle (LIT2-D-HAN) and attach the emergency spring to the handle with a light rotary movement (see also the LIT2-D-HAN instructions for use).

Fix the lead line centred in the chuck.

Make sure that the emergency sleeve is outside the chuck (proximal).

Now, slowly increase the pressure on the stone by turning the tension screw. If the emergency spiral is under tension, we recommend that you wait a few seconds after each complete rotation of the tension screw before continuing. This allows the force to build up continuously to control how the stone is crushed.

**ATTENTION!** The stone extraction baskets of the BAS series suitable for lithotripsy are equipped with a defined rated break point. Once crushing has been completed, the basket with the emergency spring is carefully removed from the duct. The stone concretions can be retrieved out of the duct with a balloon catheter or a small stone extraction basket.

## After completion of the surgical procedure

Reusable instruments must be correctly prepared. Follow the enclosed instructions for reprocessing.

If the instruments can not be used anymore, they must be disposed of in accordance with the relevant applicable hospital and administrative guidelines and in accordance with the relevant applicable legal regulations.

## Disassembly and assembly

To disassemble the instrument, unscrew the clamping grip and remove it from the instrument. Release the screw cap with a rotary movement and remove the rubber seal. Now slide the exposed pull rod into the tube. The stone extraction basket with the pulling cable can now be withdrawn from the distal end of the tube.

Take adequate precautions to prevent contamination during disassembly.

To assemble after reprocessing, slide the pulling cable into the distal end of the tube until the pull rod emerges from the proximal end. Place the rubber seal and the screw cap on the pulling rod. Fix the clamping grip onto the proximal end of the instrument, making contact with the screw cap. During this step the stone extraction basket is extended. Then fix the screw cap with a rotary movement as shown in Figure 2.

## List of all icons used on medwork products

	Date of manufacture		Use by
	Follow the instructions for use		Caution near pacemakers
	Applied part Type BF		Do not use if packaging is damaged
	Batch code		Not suitable for lithotripsy
	Item number		Contains natural rubber latex
	Packaging unit		Gastroscopy
	Do not reuse		Colonoscopy
	Sterilised using ethylene oxide		Enteroscopy
	Non-sterile		ERCP
	Manufacturer		Medical device
	Sterile barrier system		

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