**FUJIFILM**

**medwork**

## **VAL-Series\_JAZZ**

**VAL1-G7**

404450305908

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## Instructions for use

### Intended use

The VAL series biopsy valves are used to close the working channel of endoscopes. The VAL series air/water endoscope valves are used to control the insufflation and rinsing functions of an endoscope for gastrointestinal examinations. The VAL series suction endoscope valves are used to control the suction function of an endoscope for gastrointestinal examinations.

### Product features


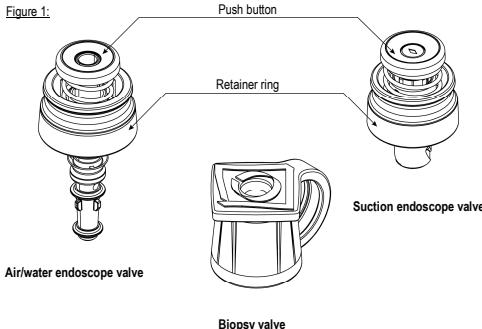
Figure 1: Positioning of the suction endoscope valve
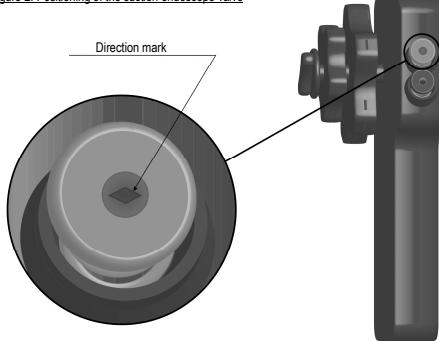
### Qualification of user

Use of these instruments requires comprehensive knowledge of the technical principles, clinical applications and risks of gastrointestinal endoscopy. The instruments should only be used by or under the supervision of physicians who are adequately trained and experienced in endoscopic techniques.

### General instructions

Use this instrument only and exclusively for the purposes described in these instructions.

**ATTENTION!** Instruments marked with this symbol are intended for single use only and are sterilised with ethylene oxide.

Sterile instruments can be used immediately. Before use, check the "Use by" date on the packaging, as sterile instruments are only allowed to be used up to this date.

Do NOT use the instrument if the sterile packaging has tears or perforations, sealing is not ensured or moisture has penetrated. All FUJIFILM medwork instruments should be stored in a dry place that is protected from light. Keep all instructions for use in a safe and easily accessible place.

FUJIFILM medwork instruments that are labelled as intended for single use only must not be reprocessed, resterilised or reused. Reuse, reprocessing or resterilisation can alter product properties, resulting in functional failure that can endanger the patient's health or lead to disease, injury or death. Reuse, reprocessing or resterilisation furthermore entails the risk of contamination of the patient or the instrument, as well as the risk of cross contamination, including the transmission of infectious diseases.

Contamination of the instrument can result in illness, injury or death of the patient.

### Contraindications

The contraindications for the VAL series correspond to the specific contraindications for oesophageal gastroduodenoscopy, colonoscopy, sigmoidoscopy, rectoscopy and endoscopic retrograde cholangiopancreatography.

### Possible complications

Infection.

### Precautions

After removal from the packaging, check the instruments for correct function, kinks, fractures, rough surfaces, sharp edges and protrusions. If you discover any damage or malfunction on the instruments, do NOT use them and inform your responsible field service contact or our branch office.

Any person who markets or uses medical devices is required to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Attention!** The air/water and suction endoscope valves are intended exclusively for use with FUJIFILM® gastrointestinal endoscopes from the series 700 and the endoscope EI-580BT.

The VAL series biopsy valves are intended for use with FUJIFILM® and OLYMPUS® gastrointestinal endoscopes.

**Attention!** Provide adequate ventilation in the room when using CO<sub>2</sub>. Not doing so may increase the CO<sub>2</sub> density and cause dizziness.

### Instructions for use

Position the biopsy valve (green) onto the working channel inlet valve of the endoscope.

Lock the air/water endoscope valve (blue) in the cylinder of the endoscope handle. Check that the valve is tightly seated and that there is no gap between the retainer ring and the endoscope. Before starting the examination check the operation of the valve by pressing the push button.

Lock the suction endoscope valve (orange) in the cylinder of the endoscope handle. Make sure that the valve is correctly oriented to the cutout. The direction mark on the push button indicates the insertion position (direction mark offset 90° to the endoscope handle, see Figure 2).

Check that the valve is tightly seated and that there is no gap between the retainer ring and the endoscope. Before starting the examination check the operation of the valve by pressing the push button.

After the examination remove the air/water and suction valve by pulling at the retainer ring (see product features).

Hold the biopsy valve close to the endoscope to pull it from the working channel.

### After completion of the surgical procedure

Instruments intended for single use, including packaging, must be disposed of in accordance with the relevant applicable hospital and administrative guidelines and in accordance with the relevant applicable legal regulations.

### List of all icons used on FUJIFILM medwork products

	Date of manufacture		Use by
	Follow the instructions for use		Caution near pacemakers
	Applied part Type BF		Do not use if packaging is damaged
	Batch code		Not suitable for lithotripsy
	Item number		Contains natural rubber latex
	Packaging unit		Gastroscopy
	Do not reuse		Colonoscopy
	Sterilised using ethylene oxide		Enteroscopy
	Non-sterile		ERCP
	Manufacturer		Medical device
	Sterile barrier system		

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